




ILC Dover Sentinel PAPR Systems and Accessories

Product Group	Part Number	Description
PAPR System - Head Cover	S-3000-01	Sentinel XL HP - Standard w/ NiMH Battery System in Duffie
PAPR System - Head Cover	S-3000-01-B	Sentinel XL HP - Standard w/NiMH Battery System in Box
PAPR System - Head Cover	S-3000-17	Sentinel XL HP w/ Alkaline Battery Pack System in Duffie Bag
PAPR System - EZ BioHood	SXL-HPEZR1	EZ BioHood System W/ NiMH
PAPR System - EZ BioHood	SXL-HPEZA1	EZ BioHood System W/ Alkaline
Conversion Kits	S-3000-05	HP Conversion Kit
CBRN Systems	S-5000-01	Sentinel XL CBRN Cap 2 - Standard w/ NiMH Battery System in Duffie
CBRN Systems	S-5000-02	Sentinel XL CBRN Cap 2 - Deluxe w/ NiMH Battery System in Backpack
CBRN Systems	S-5000-07	Sentinel XL CBRN Cap 2 - Standard w/ Alkaline Battery System in Duffie
CBRN Systems	S-5000-08	Sentinel XL CBRN Cap 2 - Deluxe w/ Alkaline Battery System in Backpack
Accessories / Spare Parts	EZ-03	EZ BioHood and head harness, 3 ea
Accessories / Spare Parts	S-4002-25	HP-P100 Cartridges (2 per pack w/plug per set) Pack of 25 sets
Accessories / Spare Parts	S-4002-3	HP-P100 Cartridges (2 per pack w/plug per set) Pack of 3 sets
Accessories / Spare Parts	S-4001-25	HP Head Cover (White) Pack of 25
Accessories / Spare Parts	S-4001-3	HP Head Cover (White) Pack of 3
Accessories / Spare Parts	S-4005-5	HP Breathing Tube - Pack of 5
Accessories / Spare Parts	BTEZM-05	Breathing Tube, Medium (for EZ BioHood), 5 ea
Accessories / Spare Parts	HHEZ-05	Head Harness (for EZ BioHood), 5 ea
Accessories / Spare Parts	S-2001	Butyl Hood Assembly
Accessories / Spare Parts	S-2002	Blower Assembly
Accessories / Spare Parts	S-2003	NiMH Battery Pack Assembly, Rechargeable
Accessories / Spare Parts	S-2009	Smart Battery Charger / Single
Accessories / Spare Parts	S-2010	Flowmeter
Accessories / Spare Parts	S-2016	CBRN Cartridges (3 each)
Accessories / Spare Parts	S-4011	Quick-Loc waist belt
Accessories / Spare Parts	S-4012	OV/AG/HE Cartridge (Set of 3)
Accessories / Spare Parts	S-4013	Alkaline Battery Pack Assembly

Sentinel XL Series PAPRs

Hoods and Head Covers

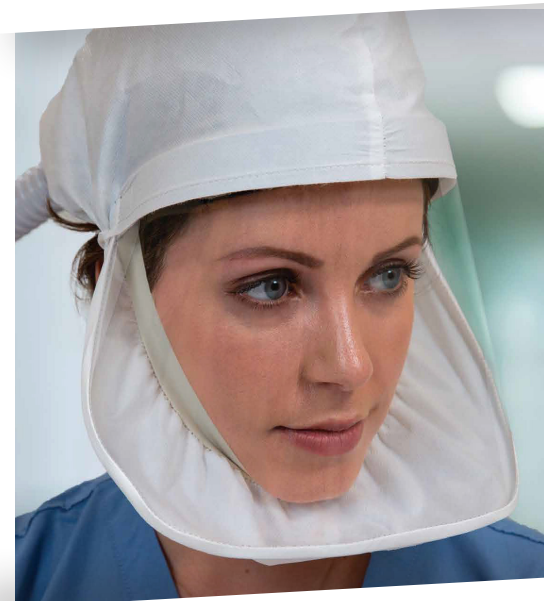

**S-4001-3 and -25
Head Cover White**



**S-2001
Butyl Hood Assembly
includes Beathing Tube**



**EZ-03, EZ-25
EZ BioHood**

FILTRATION	POWER SUPPLY	AIR DELIVERY SYSTEM	SUSPENSION
 S-2016 CBRN Cartridge (3 ea)  S-2016 CBRN Cartridge (3 ea)  S-2016 CBRN Cartridge (3 ea)	 S-2003 Battery Rechargeable NIMH S-2009 Smart Battery Charger  S-2016 CBRN Cartridge (3 ea)	 S-2002 Blower S-2010 Flow Meter  BTEZS, BTEZM, BTEZL EZ BioHood Breathing Tube (25, 33, & 39 inch) S-2005 HP Breathing Tube (33 inch)	 S-4011 Quick-Loc Waist Belt 



Is It Time to Replace Your PAPR Parts?

Consult this checklist after each use to inspect your system for any signs of damage or wear that may affect performance of the respirator and reduce the degree of protection provided.

If you answer yes to any of the below, it is time to order replacements!

Yes No

HEAD COVER

Are there tears or punctures in the hood assembly?

Inspect the stitching closely. There should be no tears that could permit contaminated air to enter the head cover.

VISOR

Are there scratches or other visual distortions that make it difficult to see through the visor?

BREATHING HOSE

Are these tears, holes, or cracks?

Carefully examine the entire breathing hose.

Is the hose becoming stiff or inflexible?

CARTRIDGES

Is there mechanical damage to the cartridge?

Have you replaced them IAW your location's change out schedule?

Need more PAPRs to go around to eliminate sharing or overuse?

ILC Dover is fully stocked with Sentinel XL™ PAPRs that are ready to be shipped.

Order yours at ilcdover.com.



ILC Dover does not recommend or endorse any specific procedures, application methods, or frequency of cleaning for our products. Implementing any protocol is the sole responsibility of the equipment end user in consideration of, but not exclusive to (i) recurrence and interval of use, (ii) single or multi-users, (iii) environmental conditions, and (iv) the type and level of potential contamination of the components involved during use.

CLICK HERE TO LEARN MORE ABOUT CARING FOR YOUR SENTINEL XL™ PAPR.